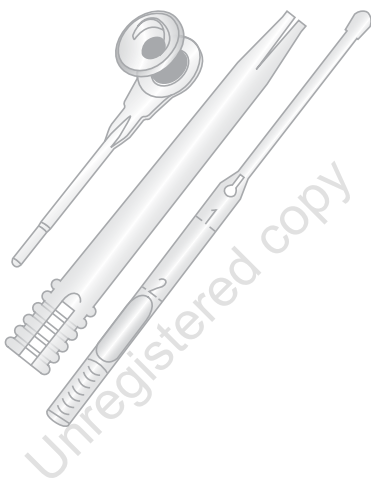
PROVOX[®] 2

Instructions for Use Patient's

Rx
ONLY

CE 0413

STERILE EO

Atos

Atos Medical Your voice

Figure 1

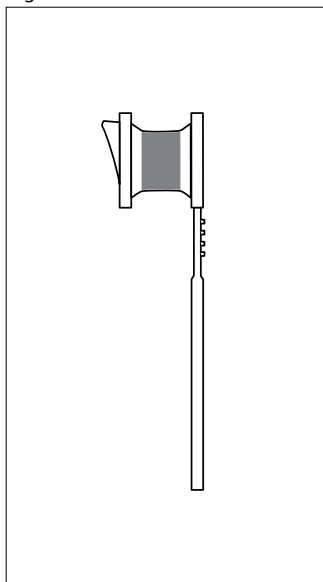
Figure 2

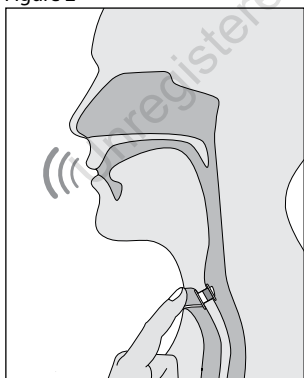
Figure 3

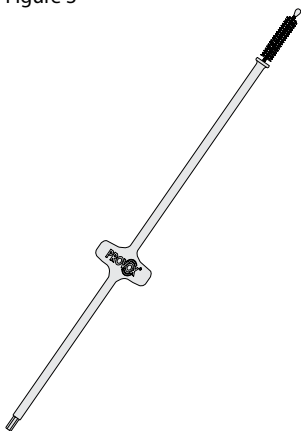
Figure 4

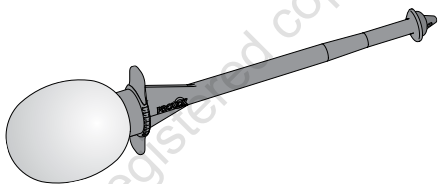
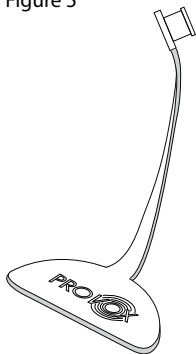
Figure 5

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox is a registered trademark of Atos Medical AB. Provox2 is protected by US patent no 5.976.151, JP 4.122.058 and other patents.

Unregistered copy

Contents

ENGLISH	6
DEUTSCH	11
NEDERLANDS.....	16
FRANÇAIS	21
ITALIANO.....	26
ESPAÑOL	31
PORTUGUÊS.....	36
SVENSKA	41
DANSK.....	46
NORSK.....	51
SUOMI	56
ČESKY	61
POLSKI.....	66
HRVATSKI	71
ΕΛΛΗΝΙΚΑ	76
TÜRKÇE	81
РУССКИЙ	86
עברית.....	94
العربية	99
ORDERING INFORMATION	100

Contents

1. Glossary	7
2. Descriptive Information	7
2.1 Intended use	7
2.2 CONTRAINDICATIONS	7
2.3 Description of the device	7
2.4 WARNINGS	7
2.5 PRECAUTIONS	8
3. Instructions for use	8
3.1 Using the Provox2 to speak	8
3.2 Cleaning the Provox2	8
3.3 Device lifetime	8
3.4 Accessories	8
4. Adverse events/ Trouble shooting by symptom	9
5. Additional Information	10
5.1 Compatibility with MRI, X-ray and radiation therapy	10
5.2 Date of Printing and User Assistance Information	10

1. Glossary

Clinician - Medical professional or properly licensed speech and language therapist / pathologist or clinical specialist nurse who is trained in voice rehabilitation procedures.

HME - Heat and Moisture Exchanger. Device that retains the heat and moisture of exhaled air, which is returned to the air when you breathe in.

PE-Segment - Pharyngo-esophageal segment. The part of the food pipe where sound is created by vibration of tissue when using a voice prosthesis.

TE-puncture – Tracheoesophageal puncture. Small artificial opening created between the wind pipe and the food pipe in which the voice prosthesis is placed.

Tracheostoma - Breathing opening in the front of the neck, where the windpipe is connected to the skin (also called ‘stoma’).

Voice prosthesis -A one-way valve that is inserted into the TE puncture. The valve has a flange on either side that allows the valve to stay in place. Air can be redirected through the valve to the food pipe for speaking, while reducing the risk of food and liquids entering the wind pipe.

2. Descriptive Information

2.1 Intended use

The Provox2 Voice Rehabilitation System is intended for use in surgical, prosthetic voice restoration after total laryngectomy. The prosthesis may be inserted by the physician at the time of the total laryngectomy (primary puncture), or at a later date (secondary puncture), or may be used to replace the present prosthesis.

2.2 CONTRAINDICATIONS

There are no known contraindications for use or replacement of the Provox2 prosthesis among patients already using prosthetic voice rehabilitation.

2.3 Description of the device

The Provox2 Voice Prosthesis (Fig.1) is a sterile single use one-way valve that keeps a TE-puncture open for speech, while reducing the risk of fluids and food entering the windpipe. It is made of medical grade silicone rubber and radio-opaque fluoroplastic. The prosthesis is not a permanent implant and needs periodic replacement. The outer diameter is 7.5 mm (22.5Fr). The Provox2 is available in lengths 4.5, 6, 8, 10, 12.5 and 15 mm.

The Provox2 Voice Prosthesis package also contains one Provox Brush with a Provox Brush Instructions for Use.

2.4 WARNINGS

Accidental swallowing of the Provox2 voice prosthesis may occur. If this occurs, contact your physician who will advise you further.

Accidental aspiration (i.e. the device falls into the windpipe) of the Provox2 voice prosthesis or other components of the Provox2 voice rehabilitation system may occur. Immediate symptoms may include coughing, choking or wheezing. If this occurs seek immediate medical treatment. A foreign body in your airways may cause severe complications and has to be removed by a clinician.

Re-use and re-processing may cause cross-contamination and damage to the device, which could cause patient harm.

2.5 PRECAUTIONS

Consult your clinician if:

- There is leakage through or around the prosthesis (coughing, and/or change of mucus color).
- Speaking becomes difficult (higher effort and/or voice sounds more strained).
- There are any signs of inflammation or tissue changes at the puncture region (pain, heat, swelling, traces of blood on the brush after brushing).

To reduce infection risk:

Make sure that your hands are thoroughly cleaned and dry before placing your hands in the area of your tracheostoma, and before cleaning your voice prosthesis and/or accessories. Keep all devices clean that may enter or make any contact with your tracheostoma and voice prosthesis.

To reduce risk of product damage:

Only use genuine Provox accessories that are intended for use with Provox2 for the handling and cleaning of your prosthesis. **Other devices may cause personal injury or damage to the voice prosthesis.**

3. Instructions for use

3.1 Using the Provox2 to speak

When you close off your tracheostoma, you can direct the air from your lungs through the prosthesis into your food pipe (Fig. 2).

That air stream causes vibration of the tissues in your food pipe, which produces the sound of your voice. You can speak by using a Heat and Moisture Exchanger like the Provox HME, using a handsfree speaking device like Provox FreeHands FlexiVoice or simply by closing your stoma directly with a finger.

3.2 Cleaning the Provox2

In order for you to speak with your voice prosthesis, it must be clean so that air can pass through it. Regular cleaning of the prosthesis also helps to prolong device life.

Brush the prosthesis with the Provox Brush twice a day and after each meal to remove mucus and food remnants from the prosthesis. Flushing the prosthesis with the Provox Flush also helps to clean the device.

3.3 Device lifetime

Depending on individual biological circumstances the device life varies, and it is not possible to predict the structural integrity of the device over a longer period of time. The material of the device will be affected by e.g. bacteria and yeast, and the structural integrity of the device will eventually deteriorate.

3.4 Accessories

The **Provox Brush / Provox Brush XL** is used to clean the inside of the prosthesis (Fig. 3).

The **Provox Flush** is an additional cleaning device that allows flushing of the prosthesis (Fig. 4).

The **Provox Plug** is a First Aid tool that temporarily blocks leakage through the prosthesis (Fig. 5).

Additional devices

Provox XtraFlow and Provox XtraMoist: Heat- and Moisture Exchangers that partially restore lost nasal functions (conditioning of the air, breathing resistance).

Provox FreeHands FlexiVoice: A Heat and Moisture Exchanger combined with an automatic tracheostoma valve, designed to allow hands-free speech.

Provox LaryTube: A silicone tracheal cannula intended to support shrinking stomas while retaining other rehabilitation devices of the Provox System.

Provox LaryButton: A soft, self-retaining silicone tracheal cannula intended to support shrinking stomas while retaining other rehabilitation devices of the Provox System.

4. Adverse events/ Trouble shooting by symptom

It is the responsibility of your clinician to inform you about risks and benefits of prosthetic voice rehabilitation and to instruct you what to do in case you may encounter adverse events. If you are not sure that you have completely understood all instructions, ask your clinician for further explanation.

Symptom: Coughing (triggered by drinking or eating) while the prosthesis is in place.

Most common reasons: Leakage through or around the prosthesis.

Measures:

1. Clean the prosthesis with the Provox Brush and if possible also with the Provox Flush. There may be food remnants preventing the valve from proper closure.

If leakage continues:

2. Observe the prosthesis while drinking. Try to identify if it leaks through the center or around the prosthesis.
 - a.) If you think it leaks through the center you can use the Provox Vega Plug while you are eating and drinking to prevent leakage. Notify your clinician that you need a new prosthesis.
 - b.) If you think it leaks around, or if the Plug does not stop the leakage, notify your clinician that you may have leakage around your prosthesis.

Symptom: Speaking becomes more and more difficult.

Most common reasons: Valve obstruction, tissue swelling in the throat (e.g. during radiotherapy).

Measures:

1. Clean the prosthesis with the Provox Brush and/ or Flush. There may be food remnants increasing the airflow resistance.
2. If that does not help: Contact your clinician immediately.

Symptoms: Pain, heat, swelling, and rash (together or alone) in the area of the prosthesis or stoma.

Most common reasons: Tissue inflammation and infections.

Measures: Contact your clinician immediately.

Symptoms: Traces of blood are seen on the brush after brushing.

Most common reasons: Tissue inflammation and infections at the esophageal side.

Measures: Contact your clinician immediately.

5. Additional Information

5.1 Compatibility with MRI, X-ray and radiation therapy

All Provox voice prostheses except Provox ActiValve can be left in the TE puncture during MRI (tested up to 3Tesla), and during X-ray and radiation therapy (tested up to 70 Gy).

5.2 Date of Printing and User Assistance Information

The version number of this Instruction for Use and manufacturer contact information is located on the back cover.

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© Atos Medical AB, 2015.