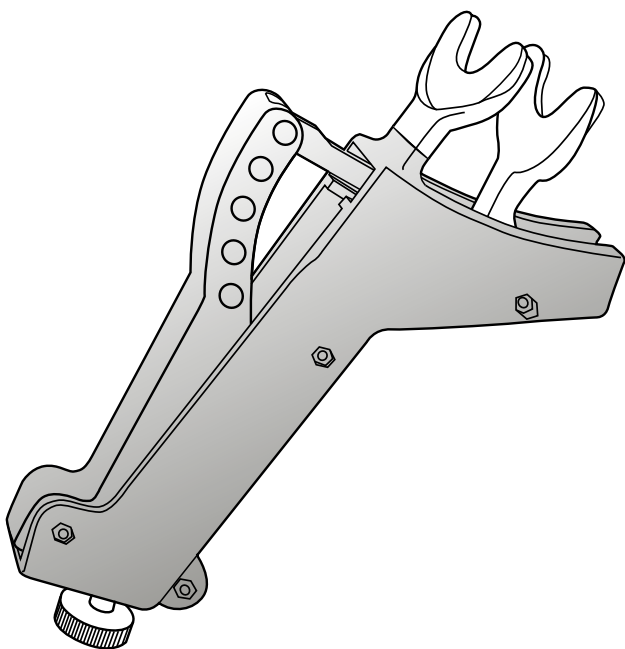
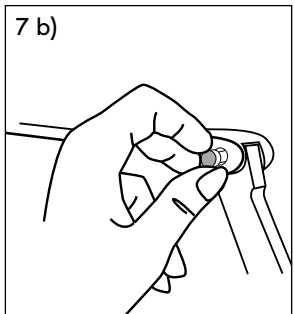
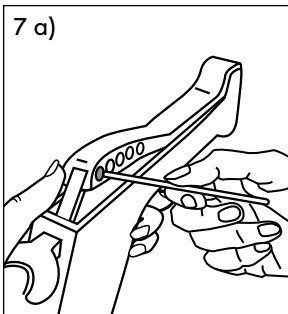
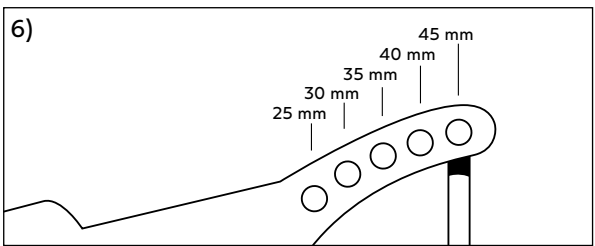
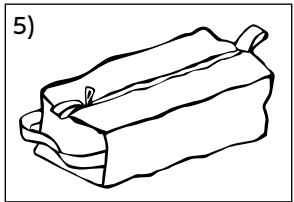
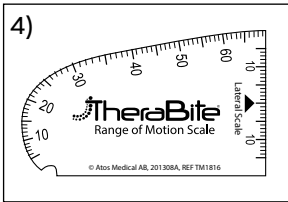
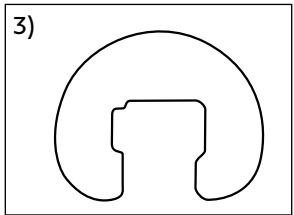
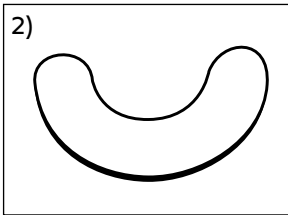
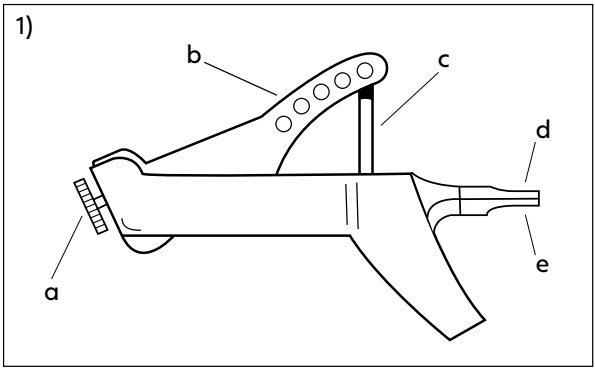
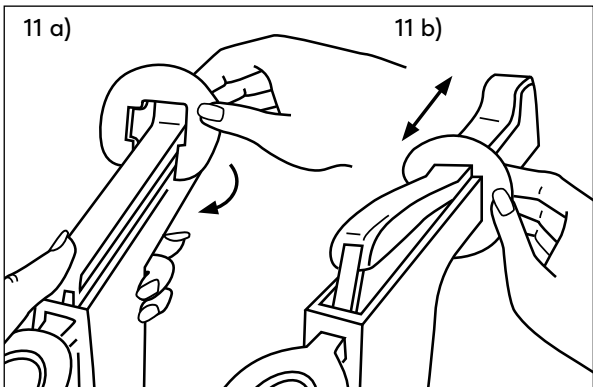
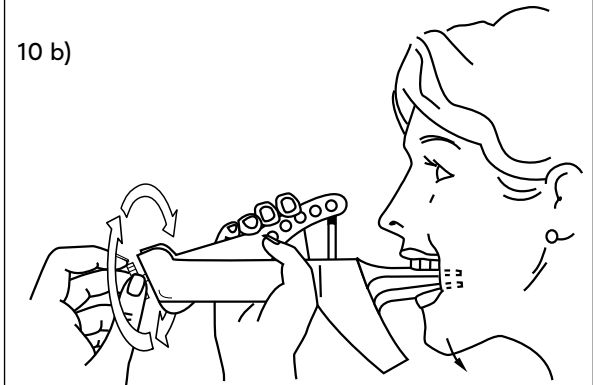
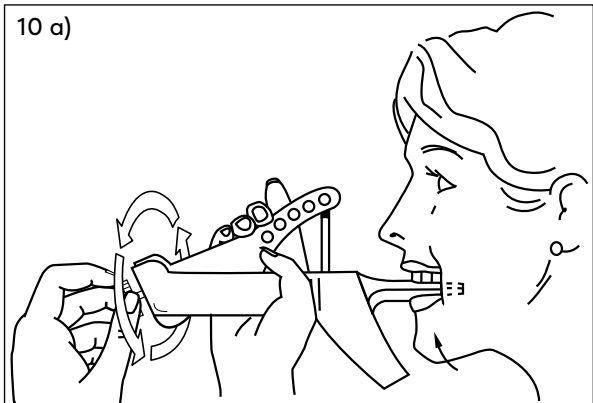
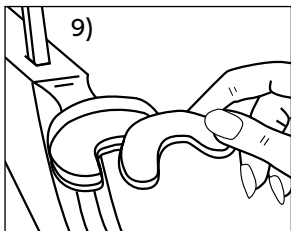
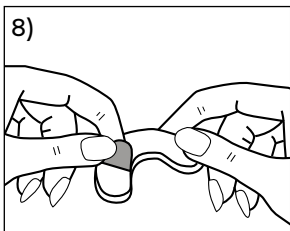
# TheraBite®

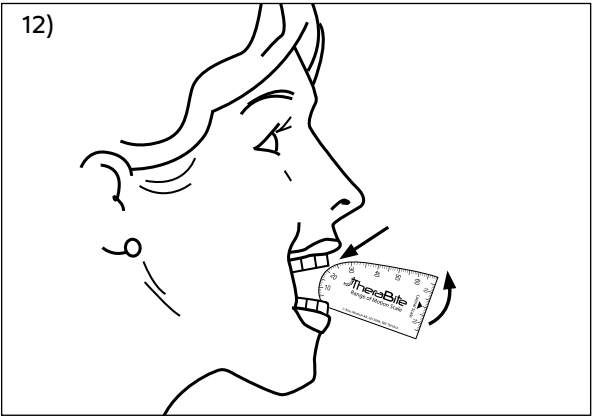
Jaw motion rehabilitation system™



12)


13 a)

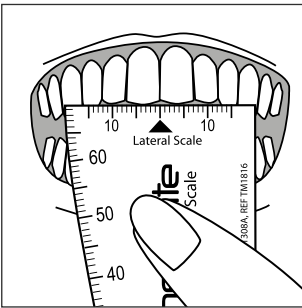13 b)

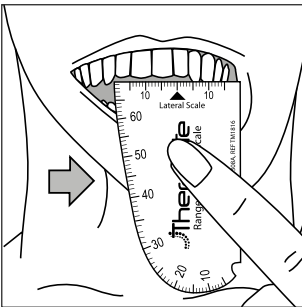13 c)

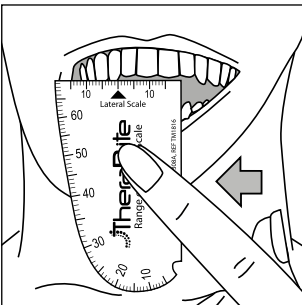
14 a)

14 b)

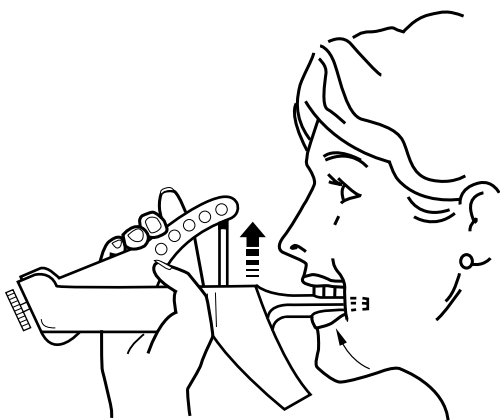14 c)

## **Disclaimer**

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Trademarks**

TheraBite® is a registered trademark owned by Atos Medical AB, Sweden. ActiveBand™ is a trademark by Atos Medical AB.

## Contents

EN - ENGLISH .....8

TR - TÜRKÇE..... 15

Before starting any rehabilitation activity involving the TheraBite system every user should consider the benefits of medically supervised programs. Clinicians may select and combine exercises to form programs that are appropriate for each user's abilities and goals, including regular monitoring and review.

## 1. Descriptive information

### 1.1 Intended use

The TheraBite® Jaw Motion Rehabilitation System is indicated for individuals who have, or are at risk of developing trismus (restrictions in their ability to open their jaw), and/or experience pain in the joints and/or muscles of the jaw. The device can also be used as a rehabilitation tool for postoperative physical therapy of the jaw, or to maintain the mouth open in a stable position, for example while performing dysphagia exercises. The TheraBite system is intended for single-patient use only.

### 1.2 Contraindications

**The TheraBite is not intended to be used by:**

- Individuals who have or may have a fracture in the maxilla or mandible (upper or lower jaw) or other weaknesses of the bones of the jaw.
- Individuals with infections of the jaw, osteomyelitis (inflammation of bone and bone marrow), or osteoradionecrosis (necrosis of bone due to radiation) of the jaw.

### 1.3 Description of the device

**The TheraBite Jaw Motion Rehabilitation System consists of the following parts:**

- 1 pc Jaw Mobilizer (Fig. 1)
- 4 pcs Bite Pads (Fig. 2)
- 1 pc Hand-Aid (Fig. 3)
- 30 pcs Range of Motion Scales (Fig. 4)
- 1 pc Carrying Bag (Fig. 5)
- 1 pc Instructions for use
- 1 pc Exercise Log

**The Jaw Mobilizer (Fig. 1)** is a hand-held device with adjustable range that is designed to help users to improve or maintain the range of motion and strength of the jaw in an anatomically correct way. Pressure applied by the user on the Lever provides either opening force for passive motion or stretching or resistance to closing for strengthening.

### **The device consists of the following parts:**

- a) Fine Adjustment Knob
- b) Lever
- c) Range Setting Arm
- d) Upper Mouthpiece
- e) Lower Mouthpiece

**CAUTION:** Do not attempt to detach the Mouthpieces from the device; this can cause failure during use.

**The Bite Pads** (Fig. 2) are self-adhesive pads intended to protect the user's teeth. They are available in three versions: Regular, Edentulous (toothless, thicker cushioning for customized trimming) and Pediatric. Clinicians may, according to their independent medical judgment, stack or trim the Bite Pads for extra protection

**The Hand-Aid** (Fig. 3) is a tool designed to hold the Lever down and maintain opening during stretching exercises.

**The Range of Motion Scale** (Fig. 4) is used to monitor the progress of the rehabilitation program. An area to record your progress is printed on the reverse side of the scale and an Exercise Log comes with your device to track your progress. The Range of Motion Scale is a disposable product and should not be cleaned.

## **1.4 Expectations of the device**

### **Passive motion and stretching with the TheraBite**

- is effective in the reduction of trismus resulting from radiation and/or scarring in head and neck cancer patients.
- is effective in the prevention of trismus in patients undergoing (chemo)radiation.
- can help to improve joint and jaw function, improve range of motion, and reduce pain in patients with temporomandibular disorders.
- has a positive effect on the outcome of surgery when used for early mobilization in patients who have undergone tempomandibular joint surgery.

The TheraBite can also strengthen the jaw muscles when resistance to closing is provided by hand or with the ActiveBand. During 'Open Swallow Exercise' the TheraBite can be used to hold the mouth open in a stable position.

**NOTE:** Prospective users should consult with their clinician before use to determine whether the TheraBite is appropriate for their individual condition.

## **1.5 WARNINGS**

- **DO NOT** continue to exercise if you feel unexpected pain at any time while exercising with the TheraBite. You should immediately stop using the device and contact your clinician.
- **DO** exercise with great caution if you have weakened teeth, gum disease, weakened bones or joints in jaw, dental plates, crowns or bridges. Injury may occur if excessive force is applied while using the TheraBite.

## 1.6 PRECAUTIONS

- Do not sterilize the TheraBite. Sterilization processes may weaken the TheraBite and it may break during use.
- Always handle the TheraBite with care; it may break during use if damaged due to rough handling.
- Do not apply too high force on the Mouthpieces. They are designed to break if too much force is applied during use in order to avoid injury to the jaw.

## 2. Instructions for use

Before the first use, the maximum opening of the TheraBite appropriate for each individual user should be determined by the clinician.

### 2.1 Preparation

#### Setting Maximum Opening of the Jaw Mobilizer

Maximum openings of approximately 25 to 45 millimeters can be set either by the position of the Range Setting Arm or the position of the Fine Adjustment Knob. The TheraBite can be used without adjusting these settings, depending on the user's ability to understand and implement instructions for the application of appropriate pressure.

#### Adjust the Range Setting Arm (Fig. 6-7)

- Hold the device on its side.
- Use a pointed object (such as a pen) to move the pin that connects the Range Setting Arm to the Lever (Fig. 7a & 7b).
- Align the Range Setting Arm in the desired position and replace the pin. (Range change is approximately 5 mm per interval.)

The scale on the Jaw Mobilizer is for reference only. Use the Range of Motion Scale for more accurate reference.

### 2.2 Operating instruction

Before each use check that the device is undamaged. Do not use the device if it is damaged.

#### Put on the Bite Pads (Fig. 8-9)

Always make sure that the Mouthpieces are completely dry before attaching the Bite Pads.

- Peel off the liner (Fig. 8).
- Attach one Bite Pad to the Upper Mouthpiece and one to the Lower Mouthpiece (Fig. 9).
- Apply pressure with your fingers to ensure proper adhesion to the Mouthpieces. Pressure and warmth under a prolonged period will make the Bite Pads stick better.

To enhance adhesion of the Bite Pads, attach the Bite Pads to the Mouthpieces before going to bed and leave them on overnight. The adhesive bond on the Bite Pads increases over time, the longer they are attached prior to use, the more strongly they will tend to adhere.

Replace the Bite Pads when they become worn. Additional Bite Pads can be ordered separately.

**CAUTION:** Bite Pads or parts of the pads that become loose during use might obstruct breathing. Always make sure that the Bite Pads are securely attached to the Mouthpieces.

### **Use the Fine Adjustment Knob (Fig. 10)**

The Fine Adjustment Knob can be used to slow down or stop the motion created by the Jaw Mobilizer. If the Knob should break for any reason, a new one can be ordered separately.

- Turn the knob all the way in (counterclockwise) to stop the Lever in the highest position (Fig. 10a).
- While squeezing gently, turn the knob out (clockwise) to allow the Lower Mouthpiece to move down slowly (Fig. 10b).
- The Lever may be released at any time without turning the Fine Adjustment Knob.

### **Use the Hand-Aid (Fig. 11)**

- Hold the Jaw Mobilizer in your left hand, and loosen the Fine Adjustment Knob so that the Lever can be squeezed all the way down.
- Take the Hand-Aid in your right hand with the narrow groove on top.
- Squeeze the Lever down, and place the open side of the Hand-Aid over the lowest part of the Lever (Fig. 11a).
- Rotate the Hand-Aid one-quarter turn clockwise, so that the handle groove is over the handle, and release the squeeze.
- To use, slide the Hand-Aid towards the mouth (Fig. 11b).

### **To measure the maximal mouth opening (Maximal Interincisal Opening) (Fig. 12)**

- Open your mouth as widely as possibly without discomfort.
- Place the Range of Motion Scale so that the notch rests on the edge of a lower front tooth.
- Rotate the range of motion scale until it contacts an upper front tooth.
- Take the reading at the point of contact.

In edentulous users, the distance can be measured between the upper and lower dentures, or if the user is not wearing dentures between the upper and lower alveolar ridges (gums).

### **To measure Lateral Movement (Fig. 13)**

- Rest the Range of Motion Scale against the chin with the upper and lower teeth gently closed together.
- Align the arrow with a space between two upper teeth (Fig. 13a).
- Move the jaw gently side-to-side and read the measurement on the Lateral Scale opposite the new position of the space (Fig. 13b & 13c).

## **2.3 Exercise programs**

It is recommended that your clinician selects and combines exercises that are appropriate for you, including regular monitoring and review.

A commonly used exercise program is 5-5-30: Five times per day. Five stretches each time. Each stretch held for 30 seconds. When short stretches are deemed sufficient a common exercise program used is 7-7-7: Seven times per day. Seven stretches each time. Each stretch held for seven seconds.

### **Stretching/passive motion**

- Adjust the Fine Adjustment Knob so that the Lever moves freely. Let the Lever come all the way up.
- Hold the Jaw Mobilizer in the hand most comfortable for you.
- Relax and place the Mouthpieces between your teeth. Your front teeth should just touch the inside rim of the Upper Mouthpiece (Fig. 14a).
- Slowly squeeze the Lever to open the Mouthpieces. Do not bite on the Mouthpieces (Fig. 14b).
- Allow your jaw to follow along with the motion of the Mouthpieces. Be careful not to tilt the Jaw Mobilizer up or down as you squeeze the Lever. Tilting may cause your jaw to move incorrectly.
- Hold open as long as instructed.
- Relax the squeeze, allowing your mouth to slowly close (Fig. 14c).

If you find it difficult to hold the Mouthpieces open, you may use the Hand-Aid.

### **Strengthening**

- Insert the Mouthpieces and squeeze until you reach the opening as instructed.
- Bite against the Mouthpieces without releasing the Lever.
- Slowly release the Lever, allowing your mouth to close against the hand pressure. Your clinician will advise you how hard to squeeze while closing.
- Rest briefly, and repeat.

For strengthening exercises using the TheraBite ActiveBand, see the TheraBite ActiveBand instructions for use.

## **2.4 Use of Exercise Log**

Use the Range of Motion Scale to size your mouth opening before and after each exercise. The exercise log is used to document and monitor your progress. The columns in the log are for the following data:

### **Section 1: Daily Exercise Log**

A: Date

B: Time

C: Type of exercise program/day

D: Sizing of jaw opening

1) Before; 2) After the exercise program.

### **Section 2: Monthly Exercise Log**

The X-axis displays the day of the month. Use the last sizing of the day from the Daily Exercise Log and enter on the Y-axis.

## 2.5 Cleaning

Use water and dish soap to clean the TheraBite Jaw Mobilizer and Hand-Aid. Rinse well, shake off excess water, and let dry. After cleaning, examine the Bite Pads and replace them if necessary.

**NOTE:** The Mouthpieces are made of a material that tends to retain moisture. Therefore make sure that the Mouthpieces are completely dry before replacing the Bite Pads and also before use.

Do not soak the TheraBite Jaw Mobilizer in any liquid or attempt to wash it in a dish washer.

Failure to follow the above may change the products properties and result in malfunction.

## 2.6 Device lifetime

Depending on handling and usage, the device life time varies. Laboratory testing of simulated daily usage for a test period of 12 month show that the device will maintain its structural integrity for this time period. Usage beyond this time is under the sole discretion of the user.

## 2.7 Additional devices

**TheraBite ActiveBand:** an elastic silicone band used together with the Jaw Mobilizer to increase and / or maintain strength and endurance of the chewing muscles.

## 2.8 Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

# 3. Troubleshooting information

### **A build-up develops on the Mouthpieces**

**Possible cause:** Build-up may be due to collected dirt on Mouthpieces after long periods of use or incorrect cleaning methods.

**What to do:** Use an emery board (nail file) to lightly file the surface of the Mouthpieces and rinse off the residue completely. Dry the Mouthpieces and apply new Bite Pads. Follow cleaning instructions.

### **Bite Pads do not stick properly to Mouthpieces**

**Possible cause:** Bite Pads may not stick properly if the Mouthpieces contain moisture or if the Bite Pads were not properly placed on the Mouthpieces.

**What to do:** Always make sure the Mouthpieces are completely dry and free of build-up before applying the Bite Pads. Apply sufficient pressure to ensure good initial adhesion to the Mouthpieces. Allow the adhesive to bond overnight if possible before use.

## **4. Additional information**

### **4.1 User assistance information**

For additional help or information please see back cover of this instructions for use for contact information.

## **5. Reporting**

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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