ENGLISH

Intended use

The Provox Capsule is a single use accessory for anterograde insertion of a standard voice prosthesis by a clinician into the tracheoesophageal puncture of a laryngectomized patient.

Description of the device

Provox Capsules are made of a water soluble material of plant origin, and sizes are available to fit four different diameters of voice prostheses, 16, 17, 20 and 22.5 Fr.

WARNHINWEIS

- Always ensure that the voice prosthesis is folded correctly and that it is properly inserted into the Provox Capsule before insertion. If the Provox Capsule comes off prematurely it could be aspirated. Note that lubricants can compromise the attachment of the capsule.
- Do not use the capsules if suspicion of contamination exists.
- Do not use Provox Capsules if the package or capsules appears damaged.
- Ensure that the voice prosthesis is functional after insertion into TE-puncture. Always follow the safety precautions specified in the instruction for use of the voice prosthesis after insertion.

PRECAUTIONS

- Do not use sharp items to insert the esophageal flange into Provox Capsule since this may damage the voice prosthesis.
- Only use a Provox Capsule of the same size or one size larger than the diameter of the voice prosthesis.

Instructions for use

- Discard the long part of the Provox Capsule and use the short part for insertion.
- Fold the esophageal flange of the voice prosthesis forward and load into the Provox Capsule.
- Make sure that the strap of the voice prosthesis is secured to the inserter pin intended for inserting the prosthesis into the puncture.
- Insert the Capsule into the puncture.
- Hold the voice prosthesis in a fully inserted position until the capsule has dissolved and the esophageal flange has completed unfolded. Slightly tug on the strap of the voice prosthesis to ensure it is in the proper position.
- Remove the inserter pin from the voice prosthesis according to instructions for use packaged with the prosthesis.

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