PROVOX Life

Coming Home

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers nowarranty of merchantability or fitness of the product for any particular purpose.

Table 1

				
Provox Life Shower		X		
Provox Cleaning Towel	X			
Provox Skin Barrier	X			
Provox Adhesive Remover	X			
Provox Life Standard Adhesives		X		
Provox Life Sensitive Adhesives		X		
Provox Life Night Adhesive			X	
Provox Life Home HME		X		
Provox Life Go HME		X		
Provox Life Night HME			X	
Provox Life Protect HME				X
Provox Life Stability Adhesive				X

Contents

EN - ENGLISH.....	4
DE - DEUTSCH.....	4
NL - NEDERLANDS.....	5
FR - FRANÇAIS.....	5
IT - ITALIANO	6
ES - ESPAÑOL.....	6
PT - PORTUGUÊS.....	7
SV - SVENSKA	7
DA - DANSK.....	8
NO - NORSK.....	8
FI - SUOMI.....	9
IS - ÍSLENSKA	9
ET - EEST	10
CS - ČESKY	10
HU - MAGYAR -	11
PL - POLSKI.....	11
HR - HRVATSKI.....	12
EL - ΕΛΛΗΝΙΚΑ.....	12
JA - 日本語	13
AR - العربية.....	13

ENGLISH

Intended use

Provox Life Coming Home is an assortment of products and information for newly laryngectomized persons. It provides guidance on product use, Pulmonary rehabilitation, and stoma care at home.

Description of Provox Life Coming Home

The Provox Life Coming Home is intended for single patient use.

The products included are selected to be used day and night after a total laryngectomy. For, contraindications, warnings, precautions and how to use the included products, see respective IFU included in the bag. Additional information can be found in Provox Life Coming Home Booklet.

Different products for different situations

In Provox Life Coming Home you will find products designed to help in different situations; Care, Day, Night and Going Out. The products can be found in the different pockets listed in table 1.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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Atos and the Atos logo are trademarks of Coloplast A/S.
© 2024 Coloplast A/S. All rights reserved.
Atos Medical AB