2020-11
10169
12.0

 TRACOE medical GmbH
Reichelsheimer Straße 1 / 3, 55268 Nieder-Olm / Germany
Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200
info@tracoe.com, www.tracoe.com

TRACOE®
comfort

CE
0 1 2 3

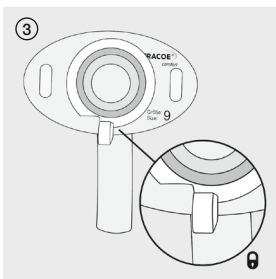
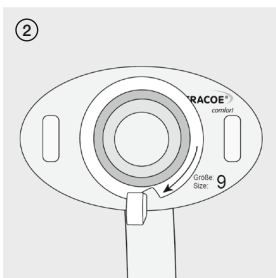
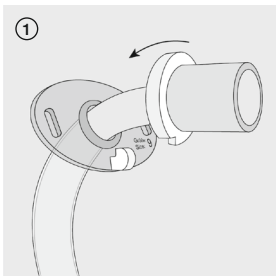
Instructions for Use
TRACOE *comfort*
Tracheostomy Tubes

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ONLY

EN / Federal (USA) law restricts this device to the sale by or on the order of a physician • **ES** /

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Instructions for Use for TRACOE® *comfort* Tracheostomy Tubes

These instructions for use contain information and instructions for physicians, nursing staff, patients and relatives who act as carers. They apply exclusively to the TRACOE *comfort* tracheostomy tubes. Please read the notes before using the product for the first time and retain the instructions for use for future reference.

Product for single-patient use: The TRACOE *comfort* tracheostomy tubes are medical products, which are destined for use with only a single patient. Multiple uses for one and the same patient are permitted.

1. General Description

The flexible transparent TRACOE *comfort* tracheostomy tubes are available in different diameters, lengths, designs and sets, including tubes for long-term use, for **tracheostomised and laryngectomised patients**. The tubes are DEHP-free and latex-free.

The shape and length of the inner cannula and outer cannula are individually adapted to each other. Therefore, you must make sure that the inner cannula is used with the outer cannula that belongs to the same set.

The size reference corresponds to the internal diameter of the outer cannula at its narrowest point (i.e. lower end) in mm. If inner cannulas are used, the internal diameter is reduced by approx. 1 to 1.5 mm.

For further details such as exact dimensions please see the current TRACOE product catalogue at www.tracoe.com.

For safety reasons, we recommend that you always have two TRACOE *comfort* tracheostomy tubes available as a replacement. Also for safety reasons, one of these tubes should have a diameter that is 1 to 2 sizes smaller than the tube that is currently being used. The smaller tube should be used in the event of problems during the reinsertion of the tube with the originally selected size.

The replacement tubes should be stored in a dry clean condition in a ventilated container. The tubes must be protected from sunlight.

Product Variants

Product variants can be manufactured according to your requested size in order to ensure the best possible functionality and comfort for the patient even for complicated anatomical and physiological conditions. For the respective order forms please visit our website www.tracoe.com.

Please Note:

Alterations to the TRACOE *comfort* tracheostomy tubes may only be carried out by employees or agents of TRACOE medical GmbH. Any alterations that have been made by third parties will result in the warranty and liability obligations becoming void with immediate effect. This applies to damage that has been caused to the products as well as any consequential damage resulting thereof.

2. Design of the TRACOE *comfort* Tube

The flat **neck flange** with eyes for fixing the neck strap is followed by the **cannula tube**, which carries the airflow and is tapered towards the tube end. This way, the insertion of the tube into the tracheostoma is made easier and the tube end does not permanently touch the wall of the trachea.

If **inner cannulas** with a perfect fit are part of the product, they are inserted into the outer cannula. **Safety aspect:** This design has the advantage that an inner cannula, which has become blocked by secretion or crust formation, can be removed from the outer cannula while the patient's airway remains secured through the outer cannula, which is still in place.

All TRACOE *comfort* tubes are delivered with a neck strap for fixing the tube and ensuring its safe position. This is a disposable product, which must not be reused.

Speaking tubes have small perforations in the outer cannula. The corresponding inner cannulas have matching bigger openings. The openings, which are positioned over each other, allow the exhaled air to flow into the upper airways.

In order to improve the phonation, the **TRACOE *comfort plus*** speaking tube has two additional holes in the inner arch in addition to the fenestration in the outer arch of the tube.

The small perforations of the outer tubes are too small for inserting a suction catheter, and this is a deliberate design in order to prevent injuries to the tracheal mucosa.

3. Intended Use

TRACOE *comfort* tubes stabilise the tracheostoma of a tracheostomised or laryngectomised patient and keep it open.

The initial selection of the tube and insertion of the TRACOE products must be carried out by a physician or by other qualified trained personnel. The users must be trained by qualified personnel in handling the products, and especially in the functioning and handling of speaking tubes and valves.

Depending on the physique and anatomy of the patient, the TRACOE *comfort* tubes or slightly longer TRACOE *comfort plus* tubes are used.

The extra-long TRACOE *comfort* tubes are used, in particular, for tracheomalacia or tracheal stenoses that are located deeper inside the trachea.

Caution: A check by a physician at regular intervals is required, especially if the patient has a tendency to crust formation and granulation tissue formation and/or heavy secretion.

4. Contraindications

CAUTION: Tubes with a speaking valve must not be used for laryngectomised patients because these patients cannot exhale the inhaled air, and there is a risk of asphyxiation.

The TRACOE *comfort* tubes are only suitable for ventilation subject to certain conditions because they have no cuff and the locking of the

inner cannula to the 15 mm connector cannot reliably ensure that the inner cannula is not dislocated. Therefore, if ventilation is required, the tube should be changed to a suitable tube such as the TRACOE *twist* or TRACOE *twist plus*.

TRACOE *comfort* tubes do not show on X-ray images because they do not have an X-Ray contrast line.

5. Tubes with a Speaking Valve

The tubes with a speaking valve are used for patients with a fully or partially intact larynx and allow the patient to speak. The inhalation is done via the tracheostomy tube. During exhalation, the speaking valve closes and routes the air through the natural airways. This way, a patient with an intact anatomy will be able to speak. It is important that the patient's upper airways are entirely unobstructed.

TRACOE *comfort* speaking tubes are fenestrated and have a speaking valve of type A or B.

TRACOE *comfort* tubes with a speaking valve type A have a flap support made of silver. The flap support can be removed from the inner cannula by pushing it outwards. For the TRACOE *comfort* tubes with a speaking valve type B, the valve support can be rotated. This mechanism allows free access to the inner lumen of the tube for both valve types, for example for sucking out tracheal secretion.

CAUTION: A physician should decide for each individual patient if a speaking valve can be inserted.

CAUTION: Patients who are unable to exhale on their own, must not use a speaking valve because of the acute risk of asphyxiation.

If the patient requires additional oxygen, the product variants with an oxygen connection can be used.

For long periods without speaking or at night, an inner cannula with a 15-mm connector or 22 mm adapter can be used, onto which a heat and moisture exchanger can be placed. The consistent use of a heat and moisture exchanger can significantly reduce crust forming.

EN 6. Tubes with a Connector / Adapter

6.1 Tubes with a 15 mm Connector

TRACOE *comfort* tubes with a 15 mm connector are used, for example, for the placement of a heat and moisture exchanger (HME), which filters, moistens and heats the inhaled air (e.g. TRACOE humid assist IV REF 642).

As an alternative to the speaking valves type A or B, a speaking valve with a silicon membrane (e.g. TRACOE phon assist REF 650-T, REF 650-TO or REF 655-T) can be placed on the 15-mm connector of a fenestrated *comfort* tube (e.g. REF 115).

Caution: In order to loosen very tight connections, such as the connection between a heat and moisture exchanger and a 15 mm connector, a suitable separating aid should always be used. Please note that the tracheostomy tube might be damaged by unsuited tools.

6.2 Tubes with a 22 mm Adapter

TRACOE *comfort* tubes with a 22 mm adapter are also used for attaching heat and moisture exchangers, for example the TRACOE humid assist I REF 640-S, and speaking valves like the TRACOE phon assist REF 650-S and REF 655-S.

7. Application of the TRACOE *comfort* Tubes

7.1 Preparation

In general, before inserting the tube into the tracheostoma, you should check if the inner cannula can be easily inserted into the outer cannula.

Please do not use any other products than the TRACOE Liquid Paraffin REF 902 for lubrication. Other oils, for example stoma oil, might stick the inner cannula to the outer cannula.

On one side of the neck flange, the neck strap (for example REF 903-F) is attached. Then a tracheal compress, for example REF 910-A or -B, is pushed over the cannula tube.

The insertion of the tube is made easier by moistening the outside of the cannula tube with a water-soluble lubricating gel.

7.2 Inserting the Tube

Hold the tube at the neck flange with one hand and spread the tracheostoma between the index and middle finger of the other hand. Carefully push the tube into the tracheostoma during the inhalation.

This step is made easier if the patient's head is slightly bent backwards. The head can be brought back to the straight position during the subsequent insertion into the trachea. Finally, the tube is secured with the neck strap by fixing the strap to the free eye of the neck flange.

Make sure that the tube is positioned free of tension in the tracheostoma. The position of the tube should not be changed by the neck strap.

The removal of the tube is made easier if the patient's head is slightly bent backwards.

7.3 Handling of the Inner Cannula

If highly viscous secretion collects in the inner cannula, which cannot be removed by means of suction and obstructs the airflow, there is a risk that the tube becomes blocked. This can result in acute danger of asphyxiation.

In this case, the inner cannula must be removed and cleaned. For this purpose, hold the neck flange of the outer cannula with one hand. With the other hand, rotate the inner cannula clockwise by a few millimetres (as seen from the patient with the inserted tube) and pull the inner cannula out of the outer cannula. The outer cannula remains in the tracheostoma where it ensures an open and safe airway.

When reinserting the cleaned and lightly oiled inner cannula (see item 8), make sure that the inner cannula is neither twisted nor bent. Push the neck flange of the inner cannula behind the little hook at the neck flange of the outer cannula by slightly rotating the neck flange of the inner cannula. This way, the inner cannula is fixed in its place (see images 1-3, model with hook at the bottom of the neck flange).

8. Cleaning and Care

The cleaning intervals heavily depend on the main illness of the patient as well as the secretion volume and consistency, any tendency to crust forming and the patient's individual needs.

At least the TRACOE *comfort* inner cannula should be cleaned daily. The respective application period of the outer cannula for the particular patient should be cleared with the physician in charge.

Caution: Always remove the tube from the tracheostoma before cleaning.

For the detailed cleaning steps please refer to the instructions for use of the below cleaning kits.

The TRACOE Cleaning Kits 930-A or -B with the Cleaning Powder and REF 931-A or -B with the Concentrated Cleaning Liquid allow the easy cleaning and care of the tubes. These kits contain a cleaning tub with a sieve insert, a cleaning agent, cleaning swabs and a cleaning brush. All of the items from the kits are also available separately.

Important: After the tracheostomy tube has been cleaned and dried, a small amount (approx. 1 drop) of the REF 902 Liquid Paraffin is applied to the outside of the inner cannula and distributed with a fluff-free cloth until a thin covering has been achieved. Then the inner cannula is inserted into the outer cannula and pulled out and pushed back in several times in order to further distribute the liquid paraffin.

Caution: Make sure that the applied amount of the liquid paraffin is as small as possible. Never apply the oil to an extent that the liquid paraffin escapes between the inner and outer cannula and enters the trachea. There is a possibility, albeit small, that permanent overdosing may result in oil pneumonia.

In many cases, the TRACOE *comfort* tube is used for out-of-hospital applications and domestic care. In such cases, and due to the germ range, which is different from that in hospitals, under normal circumstances simplified cleaning using a commonly available, domestic, mild and pH-neutral agent is possible.

Caution: The tubes must not be cleaned with aggressive domestic cleaning agents, cleaning agents for dentures or agents with a high alcohol content because these agents can impair the functioning of the products.

Caution: TRACOE *comfort* tracheostomy tubes must not be sterilised.

The tube must be checked after each cleaning and before the reinsertion. Even tubes that exhibit only minor damage must be replaced.

Disinfection

Since the upper airways are never free of germs even in healthy people, we recommend that you abstain from using disinfectants.

However, if disinfection seems necessary, you can access a list of disinfectants on the website of TRACOE medical GmbH (www.tracoe.com).

After disinfection, the tubes must be thoroughly rinsed with water in drinking quality or with distilled water and dried.

Caution: Do not use any disinfectants, which release chloride, or agents, which contain strong alkaline solutions or phenol derivatives.

You may thermally disinfect the tubes with a commonly available mild agent with a neutral pH for 5 minutes at temperatures of up to 65 °C (≤ 65 °C) without using any chemical additives. This process may result in an opaque staining, but the function of the product is not impaired.

9. Useful Life

The tube is designed so that it can be reused for a patient several times over a period of 29 days even after having been cleaned several times during that period. In individual cases, the useful life heavily depends on the patient's illness as well as the secretion and care of the tracheostomy tube. The regular replacement of the entire tube should be monitored by the physician in charge. If cared for at regular intervals, the TRACOE *comfort* tracheostomy tubes can be used for up to 6 months

10. Tube Info Card

Each TRACOE *comfort* tube contains a tube info card with detachable stickers on the front and back. These labels contain information regarding the REF no., size, length and lot no. The tube info card should be kept separately. This will make reordering easier, and the tube info card is also essential in the event of a complaint when it must be enclosed with the paperwork.

11. Storage

The products must be stored in a cool and dry place. Temporary storage for ≤ 30 days at maximum +50 °C is possible. Protect the product from sunlight and extreme cold.

12. Disposal

The TRACOE *comfort* tracheostomy tubes are made of plastic materials that are globally approved for medical applications. They contain

no latex or DEHP. These products may only be disposed of in accordance with the valid national regulations governing waste materials.

13. Returns

Returns of used products are accepted only if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website (www.tracoe.com).

14. General Terms and Conditions

The sale, delivery and return of all TRACOE products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website (www.tracoe.com).

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