TRACOE medical GmbH
Reichelsheimerstr. 1 / 3, 55268 Nieder-Olm / Germany
Tel.: +49 6136 9169-0, Fax: +49 6136 9169-200
info@tracoe.com, www.tracoe.com

TRACOE®
modular

2014-02
10410
3.0

Instructions for Use TRACOE® Occlusion Cap of the 15 mm connector of tracheostomy tubes (REF 516)

1. General Information

Sterilisation: Sterility is only guaranteed when the packaging is intact and sealed.

Single patient product: REF 516 is intended for only one patient. Multiple use on one and the same patient is allowed.

CAUTION: According to federal law (USA) the product may only be sold on medical prescription.

This product should be disposed of in accordance with the relevant national waste disposal regulations.

2. Indication
The TRACOE® occlusion cap for the 15 mm connector (REF 516) is used to assist patients in returning to normal breathing through the upper airway („weaning“).

3. Instructions for use
The TRACOE® occlusion cap (REF 516) can be placed on the 15 mm connector of an unblocked tracheostomy tube or inner cannula to wean the patient off a tracheostomy tube.

CAUTION: Before the tracheostomy tube is closed using the occlusion cap (REF 516) it is essential to ensure that the patient's upper airway is clear.

(GB)

If necessary the upper airway can be cleared by coughing or suctioning of any secretions present.

Under no circumstances should the occlusion cap be used with a blocked tracheostomy tube. Risk of asphyxiation!
When closing the tube, the patient's breathing and vital signs must be monitored by a specialist. The product should only be used while the patient is awake. If the patient shows signs of breathlessness the occlusion cap must be removed immediately.

4. Care and cleaning

The easiest way to care for the TRACOE® occlusion cap is to clean it regularly and carefully. This can be done using common household cleansers such as mild detergents.
After cleaning the occlusion cap should be rinsed with tap water or distilled water and dried.

5. Returns

Returns of used products are accepted only if accompanied by a completed decontamination certificate. You can obtain the required form either directly from TRACOE medical or from the website (www.tracoe.com).

6. General Terms and Conditions

Sales and deliveries of all TRACOE® products are carried out exclusively according to the General Terms and Conditions which can be obtained directly from TRACOE medical GmbH or from the website (www.tracoe.com).

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